

Please retain this guide for future reference. Correct cleaning and disinfection helps protect the appearance, performance and service life of newly upholstered treatment beds, examination couches and other medical furniture.

Job / Site: _____	Job No: _____
Upholstery material: _____	Manufacturer: _____
Colour: _____	Date: _____

Routine cleaning

Unless the specific upholstery manufacturer's instructions state otherwise, use warm water with a mild soap or mild detergent and a soft microfibre/cotton cloth. Remove residues with clean water and dry the surface.

Disinfection

Upholstery materials vary considerably in their resistance to disinfectants and cleaning chemicals. A product being described as suitable for PVC, vinyl or polyurethane generally should not be taken as confirmation that it is suitable for every upholstery material. Always follow the cleaning and disinfection guidance for the specific material fitted.

BEFORE USING DISINFECTANT WIPES OR CHEMICAL CLEANERS

Check that the exact product is approved for the specific upholstery material fitted. If you are unsure, contact Medi Wizard before use. This includes disinfectant wipes, bleach/chlorine products, alcohol-based cleaners, solvents and other strong chemical cleaning agents.

Products and methods to avoid unless specifically approved

- Abrasive cleaners, scouring pads or stiff brushes.
- Solvents, preparations containing solvents, or unapproved stain removers.
- Bleach/chlorine or alcohol-based products unless the material manufacturer confirms compatibility.
- Leaving cleaning or disinfecting solution on the upholstery longer than the product/material instructions specify.

Important: Incorrect, excessive or repeated use of an unsuitable cleaning/disinfection product can contribute to discolouration, hardening, cracking, peeling or other premature deterioration of the upholstery surface.

Material-specific guidance

Medi Wizard uses a range of specialist upholstery materials so that colour, performance and application requirements can be matched. Cleaning instructions can therefore differ from one job to another. Where material-specific guidance is supplied, that guidance takes priority over these general recommendations. Please contact us if you need confirmation of the material fitted or whether a particular cleaning/disinfection product is suitable.

Warranty note

Damage or premature deterioration resulting from cleaning or disinfecting products used contrary to the material manufacturer's instructions or Medi Wizard's supplied care guidance may not be covered by the applicable material/workmanship warranty.

Customer acknowledgement

I confirm that I have received the Medi Wizard Upholstery Care & Cleaning Guidelines and understand that cleaning and disinfecting products must be suitable for the specific upholstery material supplied.

Name: _____

Signature: _____

Date: _____

Job No: _____

General guidance only. Always follow the current instructions issued by the manufacturer of the upholstery material and cleaning/disinfection product. Medi Wizard can provide material-specific information on request.